Sterilization Solutions

Caronte

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FORM

REF. BIO-CHECK 98.B7
48h - 60°C

Quickly identification of processed biological indicators, thanks to the change of color of the chemical indicator on the label.

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Thanks to the 12 different positions divided on two different test benches, Caronte allows to reconcile different type of incubations with different temperature, at the same time.

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REF. **TERMOMETER**

Caronte works alone

Plan the necessary information for make the exam, and no intervention by the operator will be required, because during all the process, Caronte needs neither controls or assistance, it performs alone every necessary check.

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At the end of the incubation, no intervention by the operator will be required, except the test tube removal, as the device will proceed autonomously with:

1. Audio Alarm with which warns the operator about the end of the exam
1. Immediately print of a memo bringing specific dates and exams results
1. Automatic storage of a complete files, already done to be transfer at database.

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Caronte could be easily connected to the net and the PC, working with the software allowing a rapid exchange of data and an easily archive of the exams done guaranteeing the safety of the documents.

- Immediate exchange dates for the transfer of the performed exams, of the operator list and the sterilizers used.
- Easy, safety and rapid storage.
- Export and import of the information from usb flash drive to guarantee the maximum safety.

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Technology of reading

Ensuring the temperature's conditions and the feeding of organic spores through the incubation period, Caronte allows to minimize the necessary timings and ensures, thanks to the technology, a safe 100% results.

Thanks to a constant temperature, with a possibility of error under 2°C, to the nourishment of the spores, and to precise color of the sensor, the device perceives in little time any variation of color of the culture, individualizing with certainty which sterilizations have gone to good end and which not.

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Available also economic incubator version only for biological indicators with results in 24/48h.

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Sterilization Solutions
ECS[®]
Euro Clinical Service

E.C.S. srl
Via 1° Maggio, 18/22
23881 Airuno (LC) Italy
www.ecssrl.com - info@ecssrl.com

Certified Company / Azienda certificata • ISO 9001:2008 EN ISO 13485:2012